

Unitron Accessories and Roger™ Guide

Accessories and Roger Guide

Hearing aid technology is getting smarter and smarter. We can now enjoy a wide variety of listening environments with the help of very sophisticated accessories.

Consider these situations and think about your listening preferences:




I'd like to hear the TV clearly without cutting the sound off from others watching with me



I want to change the settings on my hearing aids, but I don't have the app and I'd rather not touch my hearing aids while I'm wearing them



I don't have a Bluetooth enabled phone, but I'd like to stream my music to my hearing aids

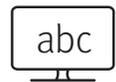

I'd like to have access to live subtitles

| | | | | |
|--------------------|---|---|---|---|
| TV Connector | ● | | ● | |
| PartnerMic | | | | |
| Remote Control | | ● | | |
| Roger Clip-On Mic | ● | | ● | |
| Roger On | ● | | ● | |
| Roger Select | ● | | ● | |
| Roger Table Mic II | | | | |
| Roger Neckloop | | | | ● |

Unitron hearing aids now allow for direct streaming from any Bluetooth® enabled device, making phone calls and listening to music more enjoyable than ever before. However, some listening environments are too challenging for hearing aids to handle on their own. That's why we have multiple devices that can help you jump back into the conversation, no matter where you are.



Noise gets in the way of my conversations in small groups around a table, like at a restaurant



I use a landline phone at home/ at work; is there something I can get to help me hear better while using the landline?



I mainly speak with one person at a time, but I can't follow along well when there's noise present



I'm most often in conversations involving multiple speakers or in big meeting spaces. I can't follow along, and it keeps me from participating

| Noise gets in the way of my conversations in small groups around a table, like at a restaurant | I use a landline phone at home/ at work; is there something I can get to help me hear better while using the landline? | I mainly speak with one person at a time, but I can't follow along well when there's noise present | I'm most often in conversations involving multiple speakers or in big meeting spaces. I can't follow along, and it keeps me from participating |
|--|--|--|--|
| | | ● | |
| | ● | ● | |
| ● | ● | ● | ● |
| ● | ● | ● | ● |
| | | | ● |

Wireless accessories

These devices are ready to use right out of the box and will connect directly to your hearing aids.

TV Connector

Operating range: 15m



With up to a 15m range, you can use the TV Connector to stream the TV signal to an unlimited number of compatible hearing aids. For loved ones that are not hearing aid users, they will continue to hear the signal as they normally would through the television's sound system – the TV Connector does not cut the sound off for others in the room.

Listening over distance can be very challenging, especially when there is noise present. Unitron wireless accessories help to bridge the gap when trying to understand speech farther afield.

PartnerMic

Operating range: 10m

Battery life on a single charge: 6 hours



PartnerMic is a great option for one-to-one conversations in busier environments. It connects wirelessly to your hearing aids. PartnerMic has a 10m range, making it ideal for chatting to your companion across a table or while out on a walk.

Remote Control

Operating range: 1m

Battery life: 12 months



The Remote Control provides a simple way adjust the volume or change between available programs.

Roger microphones

To get a Roger system up and working, we need 3 things:
a hearing aid, a receiver, and a microphone.

Roger On

Operating range: 25m

Battery life on a single charge: 8 hours



The Roger On is the newest addition to the Roger microphone family. It allows for enhanced speech understanding in poor acoustic environments, where there is a lot of noise present, as well as listening to speech at a distance. You'll have access to 3 microphone modes: table (group) mode, pointing mode, and presenter mode. These microphone modes will activate automatically based on the position of the device, or you may choose to lock it into your preferred mode by either pressing the button on the device or using the MyRogerMic App for Android and iOS. Now, with

Roger On v2, headset mode ensures that your voice gets through to the listener clearly over any noise present, while simultaneously sending the streamed signal to your hearing aids. Finally, with the Roger On and the dock, you can enable the TV Connector functionality, allowing you to listen to the television clearly without cutting the sound off for anyone else.

Roger microphones work differently than Unitron wireless accessories. They can analyze the level of noise that's around you and separate the speech sounds you want to hear from the noise that's getting in the way of your conversation. In order to form a connection, Roger microphones require receivers to be linked to your hearing aids. Additionally, Roger microphones can be linked together to form a network, allowing for larger group conversations. Please contact your hearing care provider to discuss which Roger receiver and microphone might be best for you.

Roger Select

Operating range: 22m

Battery life on a single charge: 8 hours



The Roger Select is great for group conversations around a table where there is a high level of noise present. It can be also worn around the neck on a lanyard or lapel clip for one-to-one conversations. It offers 360-degree directionality through its 6 microphone directions, where a tap of the device will mute chosen direction, bringing the focus back to where you want it to be. It will automatically move from speaker to speaker, making for a natural and comfortable listening experience. You can also plug the Roger Select's docking station into your television to enable the TV Connector functionality.

Roger Table Mic II

Operating range: 20m

Battery life on a single charge: 16 hours



When in large meeting spaces with multiple speakers, the Roger Table Mic II is an excellent option to have in our back pocket. With the remote control, you can choose the pick-up range without having to reach over and interact with the microphone. The Table Mic II can be paired with additional Table Mics as well as other Roger microphones to ensure all participants can naturally contribute to the conversation.

Roger Clip-On Mic

Operating range: 10m

Battery life on a single charge: 6 hours



The Roger Clip-On Mic is a great option for those that find themselves having difficulty in one-to-one conversations while in noisy environments. It is small enough to clip on a lapel, making it easy to take it along with you, wherever you might be going. For situations where there are a few people involved in the conversation, multiple Clip-On Mics can be paired together to form a network.

Roger receiver

Roger Neckloop

Operating range: 30m

Battery life on a single charge: 10 hours



The Roger Neckloop is an example of a receiver that's needed to connect a Roger system to your hearing aids. Simply enable the telecoil or, loop, program on your hearing aids to access the Roger signal. It can also be used to provide live transcription or subtitles.



At Unitron, we empower people with life-enhancing hearing experiences that fit seamlessly into their world. Our sound performance technology, Experience Innovations, and intuitive design work perfectly together to provide personalization and optimization. Because everyone deserves to **Love the experience™**.

© 2024 Sonova AG, or its affiliates.

unitron.com

CE 0459  Sonova AG,
Länggässstrasse 28,
CH-8072 St. Gallen,
Switzerland

sonova
HEAR THE WORLD